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INSTRUCTIONS FOR USE

MODE D'EMPLOI

Bionic Medizintechnik GmbH ... a JMS-corporation · Max-Planck-Straße 21 · D-61381 Friedrichsdorf/Germany
Phone: +49(0)6172-75760 · Fax: +49(0)6172-757610 · info@bionic-jms.com · www.bionic-jms.com

B-IFU-509/06 2018-04

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soaked wipes or disinfectants based on alcohol or quaternary ammonium compounds.

If in doubt, send the chemical details of your disinfectants to Bionic for approval.

The usage of some soaked wipes and disinfectants in combination with intensive upholstery colors may cause color residues. If in doubt, a test wipe should be conducted on a material sample, which is available upon request.

For prepared soaked wipes and disinfectants containing Hypochlorite, the concentration should not exceed 1.000 ppm. Colorings by jeans or other textiles are excluded from any manufacturer's guarantee.

⚠ Strong acidic or alkaline disinfectants or disinfectants comprising aggressive substances (e.g. peroxide, phenol) can cause irreversible damage to the stool materials.

⚠ Avoid the contact of iodine with the stool surface!

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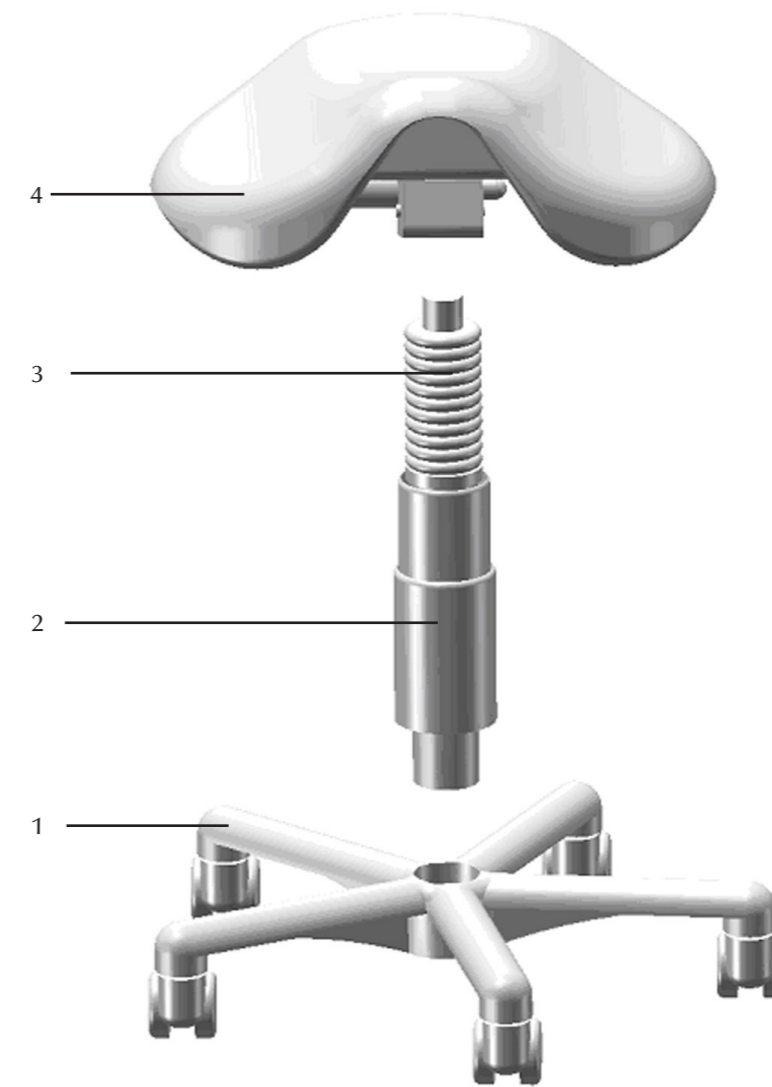
Zusammenbau / Assembly / Assemblage

A. Assembly

- Insert the gas spring [2] into the mounting hole of the base frame [1]
- Put the shell [3] over the gas spring
- Insert the gas spring into the intended hole of the seat [4]

B. Instructions for handling

1. Adjustment of height

The stool can be moved up and down by using the lever at the left side underneath the seat.

2. Tilting of the seat

The seat can be tilted to the front easily with the right lever underneath the seat.

C. Cleaning and disinfection

The stool must not be treated with automated cleaning machines or pressure washers. We recommend regular cleaning with a wet cloth and mild soap.

For regular disinfection we recommend systems with prepared

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